# #220357 – PANSEMENTS BANDAGES

FRANÇAIS

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**INFORMATION TO KEEP - TO READ BEFORE USING**

Hydrocolloid Dressings

Model	Size
Border	2x5cm, 2x6cm, 2,8x6,9cm, 5x10cm, 6x7cm, 7x7cm, 7x10cm, 7,5x7,5cm, 8x12cm, 8x12cm, 10x12cm
Non-border	2cm×4cm, 2.8cm×6.9cm, 4.2cm×2.1cm, 4.4cm×6.9cm, 4.5cm×6.5cm, 5cm×5cm, 5cm×10cm, 5.4cm×3.4cm, 5cm×15cm, 6cm×7cm, 7cm×10cm, 7.3cm×1.9cm, 9cm×14cm, 10cm×10cm, 10cm×12cm, 12.5cm×12.5cm, 15cm×15cm, 15cm ×18cm, 15cm×20cm, 20cm×20cm, Ø0,8cm, Ø1cm, Ø1,2cm

**Intended use:**

Hydrocolloid dressings are used to absorb wound exudate and provides a moist environment that supports the body's wound healing process.

**Contraindications:**

1. People who are allergic to PU, Medical Pressure Sensitive Adhesive, hot melt adhesive, sodium carboxymethyl cellulose.
2. Internal wounds, infectious wounds, wounds with damage to the deep dermis or below, chronic wounds such as ulcers, bedsores, moderate and severe burns, etc.

**Instructions for use:**

1. Before using this product, the wound surface should be cleaned or disinfected;
2. Select the appropriate product, open the package, take out the dressing and peel off the protective layer;
3. Align the center of the product with the wound surface, paste it on the skin, and smooth the surrounding area;
4. During the use of this product, please change the dressing according to the situation of the wound. Under normal circumstances, please change the dressing and replace the product within 48-72 hours; during use, if the appearance of the product changes from transparent to milky white, or is damaged or polluted, please change the dressing and replace the product in time.

**Precautions:**

1. The product belongs to disposable sterile product;
2. Before using this product, the wound surface should be cleaned and the surrounding skin should be disinfected to prevent infection;
3. The product shall not be used after expiration and the package shall not be damaged;
4. This product is disposable and cannot be reused;
5. When using, please refer to the label instructions on the package of each product;
6. In case of allergic reaction, please stop using immediately, and if necessary, please see a doctor in time;
7. When replacing this product, please pay attention to the disposal of waste according to the "requirements of hospital infection quality control"
8. Before use, please check the integrity of the single packing, if any leakage, damage, please do not use;
9. Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.
10. If the product instructions are not clear enough for the user to understand how to use the device correctly and safely, or if there is any abnormal behavior or allergy to the material during use, the user should immediately discontinue use and consult a healthcare professional

**Packaging:**

Medical paper, Single packing.

**Expiration date:**

3 years

**Production date:**

Refer to the package label

**Storage conditions:**

Products shall be stored at room temperature, avoid refrigeration and exposure to high humidity.

**Sterilization method:**

Sterilized using Irradiation Sterilization

**Description of symbols:**

	Medical device
	Marking of conformity with European directives (CE); followed by the identification number of the notified body
	Manufacturing date
	Expiration date / Use-by date
	Single use, Do not reuse
	Do not use if the package is damaged or opened
	Sterilized using irradiation
	Lot number / Batch number
	Protect from sunlight
	Keep dry
	Manufacturer Planet (Anhui) Medical Products Co., Ltd. No.2066 West Jianshe Road, Zone haute technologie, ville de Chuzhou, Anhui, Chine
	Authorized representative MedPath GmbH Mies-van-der-Rohe-Strasse 8, 80807 Munich, Allemagne
	Do Not Resterilize
	Single sterile barrier system

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Gemachtigde vertegenwoordiger MedPath GmbH Mies-van-der-Rohe-Strasse 8, 80807 Munich, Allemagne